



southwest technologies inc.

"Treating the world well"®

## Gloves:

Made in USA

**Manufactured by:** Southwest Technologies, Inc.

**Size:** One size fits most

**Cleaning Instructions:** Remove the outside cover and hand wash in cold water and mild soap. Hang up to dry.

Wipe the white inner cover with a damp (not wet) cloth using mild soap or disinfectant. Allow the product to fully air dry before continuing to use. Use the liner for patient to patient protection.

**Components:** Gel: 65% glycerine, 17% water, 17% polymer

**Inner cover on gel:** 85%Nylon, 15% Lycra

**Outside cover:** 2 piece laminate: 100%Polyurethane/100% Polyester

**Hook:** 100% Nylon

**Stretch Loop:** 30% Elastane, 70% Nylon

**Liner:** 100% Polyester

## Slippers:

Made in USA

**Manufactured by:** Southwest Technologies, Inc.

**Size:** One size fits most

**Cleaning Instructions:** Material is not removable. Wipe the outside cover with a damp (not wet) cloth using mild soap or disinfectant. Allow the product to fully air dry before continuing to use. Use the liner for patient to patient protection.

**Components:** Gel: 65% glycerine, 17% water, 17% polymer

**Outside cover:** 2 piece laminate: 100%Polyurethane/100% Polyester

**Hook:** 100% Nylon

**Stretch Loop:** 30% Elastane, 70% Nylon

**Liner:** 100% Polyester

## Caps:

Made in USA

**Manufactured by:** Southwest Technologies, Inc.

**Size:** One size fits most

**Cleaning Instructions:** Material is not removable. Wipe the outside cover with a damp (not wet) cloth using mild soap or disinfectant. Allow the product to fully air dry before continuing to use. Use the shower cap for patient to patient protection.

**Components:** Gel: 65% glycerine, 17% water, 17% polymer

**Outside cover:** 2 piece laminate: 100%Polyurethane/100% Polyester

**Hook:** 100% Nylon

**Stretch Loop:** 30% Elastane, 70% Nylon

**Shower Cap:** 100% Polyethylene

   Finger Bobs™   

1746 Levee Road, North Kansas City, MO. 64116  ph:(800) 247-9951 ph:(816) 221-2442 fax:(816) 221-3995  www.elastogel.com